

PROTEKT®

PROTEKT Grzegorz Łaszkiwicz, ul. Starorudzka 9, 93-403 Łódź, Polska
tel.: +48 42 29-29-5
www.protekt.com.pl
e-mail: handlowy@protekt.com.pl

INSTRUCTIONS FOR THE USE OF A PLATE STRETCHER DX080



Spis treści

1. Description	3
2. Dimensions	3
3. Application	3
4. Operation manual.....	4
5. Cleaning and disinfection	4
6. Warranty and warranty conditions	4
7. Periodic inspection	5


Please read this manual carefully before use. The manufacturer is not responsible for improper use of the stretcher or failure to follow the instructions contained in the manual..



1. Description

The DX080 sheet stretcher is ergonomic and easy to use. Their compatible design means that they take up little space in transport. These stretchers are used where the transportation of the casualty does not require stiffening (where there is no suspicion of damage to the spine or pelvis). The stretcher is equipped with eight handles located on both long sides. The stretcher is made of plavil with high resistance to damage and abrasion.

2. Dimensions

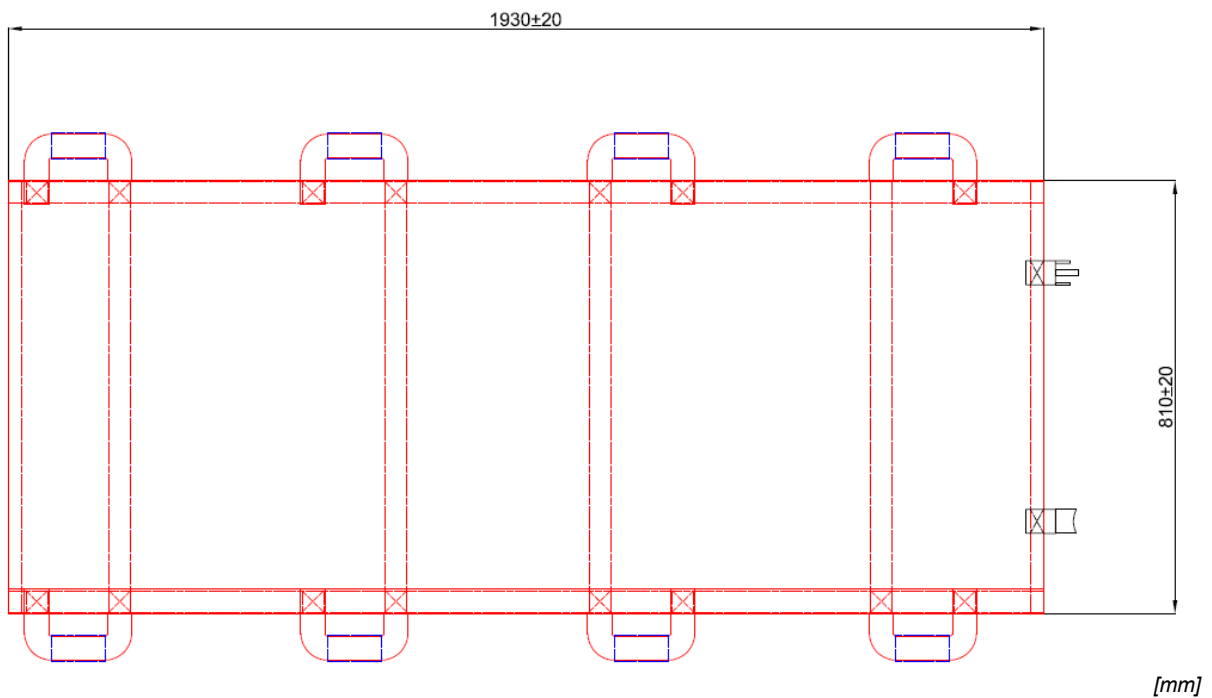

3. Using

The DX080 sheet stretcher must be placed on even surfaces. Cross straps (reinforcements) should be placed on the underside. To ensure correct patient transport, the patient must be placed in the center of the stretcher.

4. User manual

The DX080 sheet stretcher should be carried by at least 4 people. Rescuers should be placed evenly around the stretcher. This will avoid unnatural or tiring patient position or deterioration of the patient's health during transport. After leaving a hard-to-reach area, the patient should immediately be placed in a suitable means of transport (vacuum mattress, board stretcher, etc.).

5. Cleaning and disinfection

For cleaning and quick disinfection, it is enough to wash the stretcher with a mild soap-based detergent. Plavil sheet stretchers are easily washable with a cleaning agent up to 60°C.

For disinfection, use an alcohol-based disinfectant, 62 (e.g. MERIDA DESINFECTIN COMPLEX M430 PLUS)

After cleaning and / or disinfection, the stretcher must be unfolded and hung up to dry. Curtain stretchers should not be dried in the dryer or in centrifuges. The stretcher should be completely dry before it is folded, stowed away and used again. Mainly attention should be paid to the textile tapes used in the product.

6. Warranty and terms of warranty

The manufacturer gives a 12-month warranty for the DX080 stretcher. The manufacturer ensures that the product is free from material defects and those arising in the production process.

The manufacturer is responsible for the safety, reliability and operation of its products only if:

- maintenance services, repairs, adjustments and modifications are performed by the manufacturer or persons authorized by it,
- the product has been used properly, strictly in accordance with the instructions for use, in accordance with the terms of this warranty, the manufacturer shall not be liable for incidental or consequential damages as well as resulting from unauthorized repairs or modifications, as well as caused by random events or improper use of the product or its part .

The warranty does not cover the effects of normal wear and tear as well as stains, discoloration, fading or other minor damage that do not reduce the usability of the product.

7. Periodic inspection

- Periodic inspection (at least once a year) should be performed by a competent person (from PROTEKT or an authorized service of the manufacturer).
- In case of ambiguity, please contact the manufacturer, PROTEKT.

Control activities:

- Inspection of cracks, abrasions, material damage
- Control of completeness and presence of plastic elements
- Control of the reinforcement tapes
- Seam inspection

